



EZ Way, Inc.

EZ4 Competency Checklist

Purpose: To assist in the proper training of operating the EZ4 Multi-Purpose Stretcher-Chair.

Staff Name: _____ Date: _____ Observed by: _____

Yes No

- a) Demonstrate brake and steer caster system operation.
- b) Explain the purpose of the red sticker on the steer caster.
- c) Demonstrate side rail lock and release operation.
- d) Demonstrate motorized positioning by hand-held pendant control.
 - Back Section
 - Height Adjustment
 - "X" Seat Tilt (if equipped)
 - Leg Section
 - Auto Contour
 - "T" Tall Column (if equipped)
- e) Demonstrate push bar operation.
- f) Demonstrate knowledge of the radiolucent back section - X-Ray/Fluoro/C-Arm compatible (if applicable).
- g) Perform the procedures for "B" battery or "A" A/C plug-in and charging.
- h) Perform the quick-release backrest operation with the back extension mounted on the chair.
- i) Demonstrate knowledge of all the equipped options.
 - "W" Wide Width
 - "Q" Scale
 - "F" Manual Folding Foot Rest
 - Other: _____
 - "R" Folding Upper Rails
 - "G" Removable Head Section (ENG)
 - "E" Export Plug-in
- j) Demonstrate knowledge of any optionally equipped accessories you have selected.
 - Belt
 - X-Ray Cassette Holder
 - Other: _____
 - Telescoping I.V. Pole
 - Oxygen Tank Holder
- k) Demonstrate knowledge of the foot rest operation and warning (if equipped). ~ DO NOT STAND
- l) Demonstrate proper procedure for seating a patient (ingress/egress on the side of the seat).

* EZ Way recommends that the facility fully train staff on proper operation of the chair PRIOR to utilizing with a patient.

* Review all "Notices, Cautions, Warnings and Dangers" in Operations Manual.