



EZ Way, Inc.

# Twin-Turner User Manual

## (500943 Bi-directional Turning Sling)




This sling, designed for use with a ceiling lift or mobile lift, is used for turning a patient onto their side.



### STEPS 3-4

Fold the other corner into the middle and over the top of the first corner.

While holding onto the folded corners, use the other hand to grab toward the middle of the sling, ensuring that the straps are tucked underneath the fabric.

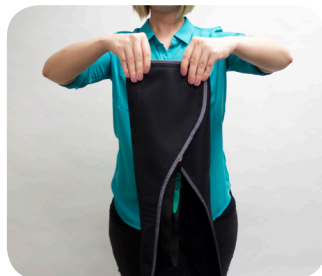

### STEP 8


Walk to the side where the patient will be turned, and using the Velcro strap, attach the sling securely to the bed or other solid attachment point located on the frame bed. Do not attach to the side rails.

Make sure the bed rails are raised on the side of the bed where the patient will be turning.



### STEPS 1-2

Holding twin turner upright and Velcro strap facing away from you (out), fold one corner into the middle of the sling.

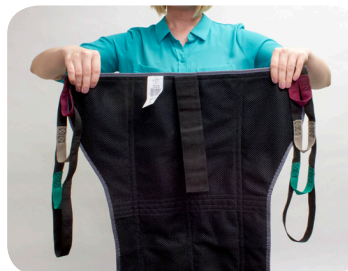

### STEPS 5-7

Place your hand (palm down) onto the bed and on top of the folded portion, and slide the folded portion of the sling under the small of the patient's/resident's back. (For ease, it may be beneficial to have a second person depressing the bed on the opposite side.)

Walk around to the opposite side of the patient/resident and pull out the folded portion of the sling. Unwrap the folded portion of the sling once it has been pulled through. Make sure the patient's/resident's arms are not underneath the sling. Placing the hands and arms on the chest with help insure they are not under the sling.



### STEPS 9-10

Walk back to the opposite side of the bed and attach the sling loops to the hanger bar.

Activate the lift to slowly turn the patient/resident onto his/her side. If needed, support the head with a pillow.

