




EZ Way, Inc.

*"Your Partner In
Safe Patient Handling"*

eZ Effort® Lift

Cleaning Instructions

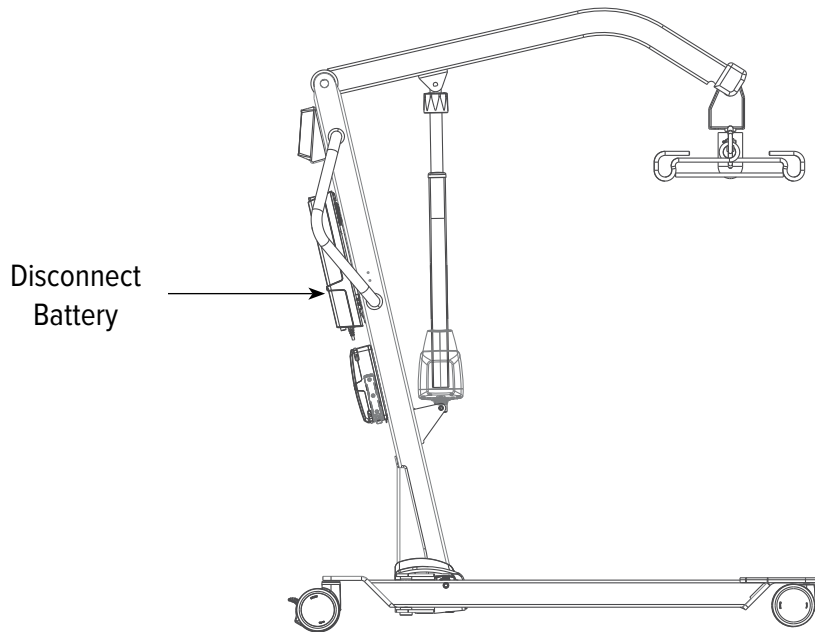

Frame:

To keep your EZ Effort clean and in good condition, we recommend using a standard germicidal spray, Sani- Wipe, or similar product.

Guidelines:

1. Disconnect battery from lift prior to cleaning the lift. Do not spray cleaning liquids directly onto the battery. Spray on cloth and wipe off battery or use a germicidal wipe.
2. Spray the cleaner onto a cloth or paper towel. Wipe the unit with the cloth or paper towel.
NOTE: Do not spray the cleaning product directly onto the machine.
3. Be careful not to wipe off the model and serial number sticker (located on the side of the mast).

Optional Sling Bag Cleaning:

► General Cleaning:

- Soils or Stains: Use neutral soap suds and lukewarm water. Do not use harsh cleansers, solvents or detergents.
- Hard-to-Clean Spots: Use standard household/ vinyl cleansers and a soft bristle brush and gentle agitation on troublesome spots or stains. Pre-soak heavy, dried-on soil.

► Disinfecting:

- All Sure-Chek fabrics may be cleaned with a 1:10 dilution of household bleach containing 5.25% hypochlorite to water, as recommended by the Center for Disease Control in Atlanta, GA.

► Specific Commercial Disinfectants:

- There are many popular disinfectants being used in care facilities. Cavi-Wipes and Virox5 are two of these that have been tested and found to be acceptable when used according to manufacturer's recommendations.

Consult the EZ Way Wipeable Fabric Cleaning Instructions for a full list of acceptable cleaners.