



EZ Way, Inc.
701 E. Washington St.
P.O. Box 89 • Clarinda, IA 51632
800-627-8940 • Fax 712-542-1899
www.ezlifts.com

500200 Wrist Strap Instructions

The wrist strap can be used on the EZ Stand to hold the arm of a stroke patient in position during a transfer.



1. The strap should be attached to the lower handle of the EZ Stand before attaching the harness to the machine.



2. Slide the strap over the lower handle of the EZ Stand arm with the white Velcro patch facing the inside of the arm (toward the patient). The wrist strap should be slid all the way to the bend in the lower handle.

3. Following the EZ Stand operating instructions, attach the EZ Way harness to patient and then attach harness loops to the EZ Stand.
4. While the EZ Stand arm is in the down position, assist patient and place hand on lower handle of the EZ Stand arm.
5. Secure wrist by wrapping free end of wrist strap around the patient's wrist and fasten loose end to the white Velcro patch.



6. After the transfer is complete, remove the wrist strap from the patient's wrist.