




EZ Way, Inc.

EZ6 Quick Notes

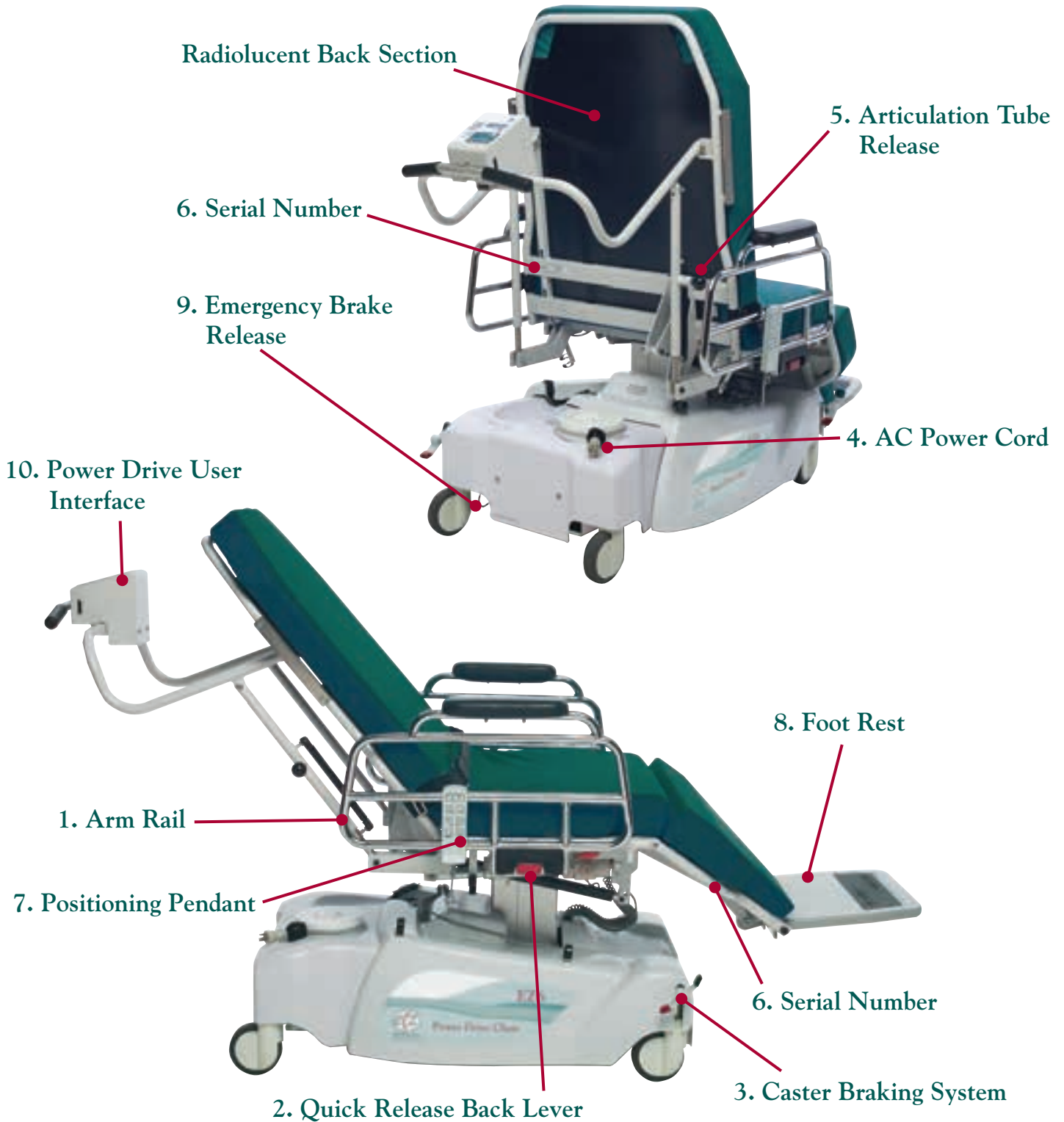

**Lock Casters and Lower Rail as
PATIENT MUST ENTER SEAT FROM SIDE.
Then raise rail up and secure lap belt around patient.**



EZ Way, Inc.

EZ6 Quick Notes

1. Arm Rail – Lowering/Raising



Lowering the Arm Rail

Grasp the top of the arm rail and **push in slightly, while pulling out on the rail release tab**. Once the arm rail latch is deactivated, the arm rail can be lowered into the down position under the seat. Once under the seat, push in on the arm rail to engage the arm rail retainer.



Raising the Arm Rail

Lift up on the arm rail until it is in the up position. The arm rail latch will engage automatically.

NOTE: Pull on arm rail after it is raised to ensure it is securely locked.

2. Quick Release Back Lever



Quick Release Back Lever

Located under the seat section, on the patient right side. To activate the release, **pull out lever**.

NOTE: The back section quick release is to be used for emergency purposes only.

3. Caster Braking System Operation



Brake Mode

The braking system is activated by **pressing the red end down** on any of the brake pedals located on the chair base. When activated, this mode will prevent all four casters from swiveling and the caster wheels from rolling.



Steer Mode

The steer lock system is activated by **pressing the green end down** on any of the brake pedals located on the chair base. When activated, the caster on the patient's right side, near the foot end will lock parallel with the base but the wheel will still roll. The other three casters will swivel and roll freely.



Neutral Mode

The braking system can be placed in the Neutral Mode by ensuring that **the brake pedals located on the chair base are in a horizontal position**.

During power drive the casters must be in the neutral position.

4. On-Board AC Plug-In

Charging the Battery

The EZ6 features on-board batteries with 10 ft. retractable AC power cord.

Best Practice: Plug into properly grounded medical grade outlet whenever Stretcher-Chair is not in use or set a daily re-charge schedule to assure unit is charged and ready when needed. Prior to use, it is recommended to check battery level. If battery is fully depleted, it requires 10 hours to recharge. Avoid repeated full depletions as it will significantly reduce the life of the batteries.

NOTE: POWER DRIVE NOT FUNCTIONAL WHEN CHARGING.





EZ Way, Inc.

EZ6 Quick Notes

5. Articulation Tube

For access to the Radiolucent Back Section for X-Ray, C-Arm or Fluro, Raise the Articulation Tube

Pull black knob on the right side of the articulation tube and gently raise. To restore, pull gently down until knob pops back into place.

The articulation tube can also be raised for smaller footprint and storage.



6. Serial Number

Locations

The Serial Number tags are located on the back section of the chair and on the back of the leg section.

NOTE: You will need to provide your chair's Serial Number when calling for service or with questions about your EZ chair.



8. Foot Rest



Positioning

Auto-extending foot rest (goes flat in the supine position). Does not fold up for patient protection.

WARNING: DO NOT STAND OR SIT ON FOOT REST. ENTER CHAIR SEAT FROM THE SIDE

9. Emergency Brake Release



Located behind the back, left wheel. Push forward to lock. Pull back to unlock.


CHAIR WILL NOT DRIVE WITH EMERGENCY BRAKE RELEASE PULLED (UNLOCK). IF POWER DRIVE IS ATTEMPTED, ALL THREE MAX SPEED SETTING LIGHTS WILL FLASH BUT THE CHAIR WILL NOT DRIVE.

7. Positioning Pendant

Pendant Control

The pendant is normally stored on top of the back section or on the side rails.

NOTE: The pendant control is NOT INTENDED FOR PATIENT USE.

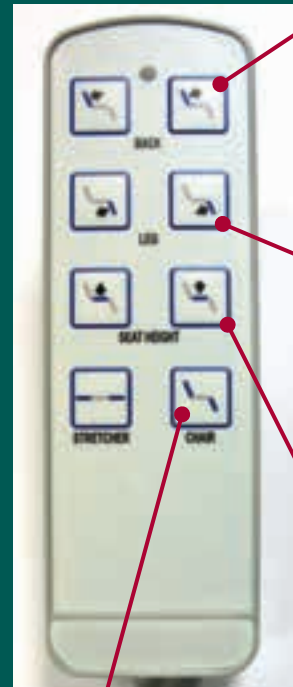

Back Adjustment

The first row of buttons on the pendant control the back section recline level. Press the left button to raise the chair back or the right to lower the back as indicated by the arrows.

Leg Adjustment

The second row of buttons on the pendant control the leg section incline level. Press the left button to raise the chair leg or the right to lower the leg section as indicated by the arrows.

Height Adjustment

The third row of buttons on the pendant control the height of the chair. Press the left button to raise the chair or the right to lower the chair as indicated by the arrows.

Simultaneous Leg & Back Adjustment (Auto Contour)

The fourth row of buttons on the pendant control the simultaneous activation of the leg and back sections. Press the left button to raise the chair leg and lower the back, putting the chair in a flat position. Press the right button to lower the leg and raise the back section, placing the unit into the chair position.



"Your Total Patient Lift Solution"

EZ Way, Inc.

PO Box 89, 710 E. Main • Clarinda, IA 51632
1-800-627-8940 • www.ezlifts.com



EZ Way, Inc.

EZ6 Quick Notes



10. Power Drive User Interface

To Operate Power Drive:

- If transporting, patient must enter and exit the chair seat from the SIDE and have Lap Belt fastened.
- Lock side rails in UP position.
- Position Chair with Pendant for proper visibility when driving.

User Interface:

- Press 'ON/OFF' button (hold until lights come on).
- Enter 4-digit code. (See department manager for code).
- Press 'Enter' button (code accepted light on).
- Set SPEED to 'Slow'.
- Press Motion Alarm (optional). Green light is illuminated when Alarm is on.
- Push Power Drive (wait for center drive wheels to lower to the floor). Green light is illuminated when drive wheels are engaged.
- Set Caster pedal to NEUTRAL position. Casters Unlocked light is illuminated when Power Drive is on.
- Rest both hands on bars located on either side of the user interface (sensors ensure hands are in position).
- To drive FORWARD – hold bar and walk forward.
- To drive BACKWARD – hold bar and walk back.

Best Practice!

- RAISE POWER DRIVE AND TURN POWER DRIVE OFF BEFORE GOING INTO ELEVATORS AND SMALL ROOMS.
- PLUG CHAIR IN WHENEVER NOT IN USE TO PRESERVE THE LIFE OF THE BATTERY!
- Power Drive will not work if chair is plugged into outlet.
- If amber Service Required light turns on, raise power drive, turn chair off, contact service.

To Stop Chair from Moving in Power Drive:

- Stop walking.
- Remove hands. Chair will come to a stop and internal brake will engage.
- Lock casters.

To Turn Off:

- Lock casters by pushing red pedal DOWN.
- Push Power Drive button (light turns off and wheels raise).
- Press 'ON/OFF' button (hold until all lights turn off).
- Plug chair into properly grounded medical grade plug when not in use to charge battery. Battery lights will turn on to show charge status.

CAUTION: Turn the power drive mode OFF before performing any procedures.

How to Operate Chair in Manual Mode if Drive Wheels are Down and Battery is Empty:

- Release Red Emergency Brake Lever by pulling it BACK toward you. (Located next to the back, left caster wheel, under base cover.)
- Be sure Caster Pedal is in NEUTRAL position.
- Push chair to a medical grade outlet, lock caster brakes and plug in using cord at the base of the chair.