



EZ Way, Inc.

EZ Way Single Patient Disposable Slings and Harnesses

Slings

Medium - #SPS200

Large - #SPS300

Small with Head Support - #SPSHD100

Medium with Head Support - #SPSHD200

Large with Head Support - #SPSHD300

X-Large with Head Support - #SPSHD400

XX-Large with Head Support - #SPSHD500

Harnesses

Medium - #SPH200

Large - #SPH300

X-Large - #SPH400

EZ Way recommends the Single-Patient Disposable Products be used as follows:

- Only for use as a single-patient accessory. Product should not be recycled or redistributed.
- Do not launder. If the accessory is laundered, it should be discarded.
- The accessory should be replaced at any sign of wear, tear, or soiling.
- Single-patient accessories are not designed for long-term use.
- Operator must inspect accessory prior to each use.
- Medium, Large, and X-Large Single-patient slings and harnesses are capable of bearing a 660 lb. weight load, and XX-Large Single-patient slings are capable of bearing a 1,100 lb. weight load but must only be used to hold the amount of weight dictated by the capacity of the lift. Lift and sling should be used by trained personnel in accordance with operation manual, video, and training checklist to avoid injury to patient.
- The single-patient sling can be used for bathing, showers, and also in pools. Dispose of sling after these activities and replace with a new sling.
- Single-patient accessories are 100% breathable.
- Made from a durable yet comfortable polyester material.