



EZ Way, Inc.

# EZ Way Floor Lift & Stand Cleaning Guide

To keep your EZ Way floor lifts and stands clean and in good condition, we recommend that you use a standard germicidal spray, Sani-Wipe, or similar product and that you follow these guidelines:

- DO NOT SPRAY PRODUCT DIRECTLY ON THE MACHINE.
- Spray the cleaner onto a cloth or paper towel then wipe the unit to clean it.
- The germicidal spray, Sani-Wipe, or similar product can be used on the control panel and front panel graphics. If not using a wipe, make sure to spray the cleaner onto a cloth or paper towel then wipe the unit to clean it.
- Be careful not to wipe off the model and serial number sticker located on the side of the mast.

EZ Way, Inc. • (800) 627-8940 • Fax (712) 542-1899 • [www.ezlifts.com](http://www.ezlifts.com)

**⚠WARNING:** Cancer and Reproductive Harm - [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).