

HARRIS MEDICAL EZ TRANSPORT CHAIR
RH3500B
Preventative Maintenance Checklist

Equipment checks to be done by user to ensure properly functioning EZ Transport Chair.

1. Grease wheels every 6 to 12 months, or as needed
Wheels have grease fittings
2. Check wheels in and around wheel axle for debris and clean.
3. Check axle adjustment. Tighten down wheel until it no longer spins freely and then back off slightly. This will ensure the wheel is tightened properly.
4. Check cables and adjust accordingly every 6 months. Cables with use can stretch and this will not allow the shock to engage. Adjustments can be made at the handle by loosening nut and threading the cable end either in or out to make it tighter or looser. This should be done every 3 to 6 months or as needed.
5. Ensure that all upholstery is in proper condition (no rips or tears). And check that all padding pieces (headrest, arm rest pads, etc...) are with the chair. This should be done before each use.
6. Ensure that all locking pins for safety strut bars and attached to the chair and in good working order. This should be done before each use.
7. As stated in cleaning instructions, it is important not to use straight bleach on the upholstery. Doing so will prematurely wear out the material. Never use a concentration stronger than 11 parts water to one part bleach (based on the 8.25% industry standard). Always follow manufacturer's instruction on label. Harris Medical recommends using a Quaternary cleaner which doesn't have the same oxidizing characteristics as bleach.

For service and sales please contact:
EZ Way, Inc.
PO Box 89
Clarinda, IA 51632
Telephone: 800-627-8940 press 3 for Service
Email: sales@ezlifts.com
www.ezlifts.com