



EZ Way, Inc.

EZ Way Roll Board Usage Instructions

The EZ Way Roll Board was designed to facilitate quick, easy lateral transfers between two parallel lateral surfaces, i.e. bed to stretcher, stretcher to OR or radiology table, etc. The inner board provides a comfortable transferring surface and has a moisture impermeable Barrier Flex covering. The rolling sheet is removable and washable, and the end cap snaps are non-ferrous and able to be used in MRI settings.



Transferring Patients with the EZ Way Roll Board

- Lock the wheels of any surface involved in a transfer, i.e. bed or stretcher and raise the bed rails on the exterior sides of each surface. The two interior surfaces should be the same height, should not be more than two inches apart, and ideally should be touching.
- Log roll the patient away from the surface they are being transferred to and slide the board under the patient and draw sheet so at least 1/3 of the board is under the patient.
- Support the patient's head and/or feet as needed.
- EZ Way recommends that two caregivers be involved in the transfer.
- The caregiver at the side of the surface the patient is being transferred to should grasp the draw sheet and pull while the caregiver on the opposite side nearest the patient gently pushes the patient. The EZ Way Roll Board will roll under the patient, providing an easy transfer.
- Once the transfer is completed, a caregiver should log roll the patient and remove the EZ Way Roll Board.

Cleaning Instructions

- A standard germicidal cleaner should be used to wipe down the inner Roll Board core and end caps.
- The sheet can be laundered and tumble dried. Do not use bleach on the product.

Manufactured & distributed by:
EZ Way, Inc.
701 E. Washington St.
P.O. Box 89
Clarinda, IA 51632
www.ezlifts.com

1-800-627-8940 or (712) 542-5102
Fax (712) 542-1899

Form #2-195
Rev. 7-27-18