

EZ2000 Shower Trolley Competency Checklist

Standard: Following procedural review and demonstration of skill, the employee will be able to safely use the EZ2000 Shower Trolley to perform patient hygiene tasks.

Method of Verification: Return Demonstration

Staff Name: _____ Date: _____ Observed by: _____

1. EZ2000 Shower Trolley weight capacity is 330lbs.
2. Only qualified staff that has completed competency training may operate this device.
3. Inspect device prior to use. If problem(s) identified, remove from service until repaired.
4. The patient must be positioned comfortably on the device. Side rails must be raised and SECURELY locked whenever patient is aboard the EZ2000 Shower Trolley.
5. Never leave patient unattended when lying on the EZ2000 Shower Trolley.
6. Patient must remain supine (lying down) on the device. Sitting on the device is not advised.
7. Brakes must be locked on all surfaces (EZ2000 and second surface) before patient is moved on or off the EZ2000.
8. Ensure EZ2000 Shower Trolley can be raised or lowered without pinching patient.
9. Operate the Trolley carefully when moving patient through halls.
10. Trolley must be cleaned after each use. Wipe underside of stretcher to prevent water from dripping onto floor.

#	Competency	Yes	No	N/A	Comment
Patient Handling Procedures for the EZ2000 Shower Trolley					
1	Explain procedure to patient. Make sure patient can tolerate log rolling.				
2	Ensure both transfer surfaces are locked, and positioned right next to each other- without gaps.				
3	Following Safe Patient Handling and Movement Algorithm guidelines, transfer patient onto Trolley using the recommended number of staff (2 minimum).				
4	Raise and secure rails. Position patient in the center of the Trolley. Ensure patient will not be pinched during Trolley use.				
5	Transport patient safely to shower room.				
6	Position Trolley drain over floor drain, sink, or toilet. Lock the device by depressing locking castor brakes until they click into position. Make sure device is secured.				
7	Attach the drain hose and drain plug. <u>Bathe patient by following facility policies and procedures.</u>				
8	Rinse patient and drain trolley by using Hydraulic Pedal and Gas Springs to lower foot end of trolley a maximum of 4 degrees. Remove body fluids and feces immediately to prevent staining of device.				
9	Keep patient warm. Level trolley. Insert plug into drain. Fold drain hose upwards to prevent dripping. Wipe underside of device to dry and prevent dripping.				
10	Return patient to room. With surfaces locked, transfer back to receiving surface following Algorithms. Raise side rails on receiving surface. Clean Trolley.				
Operating the EZ2000 Shower Trolley Hydraulic Pedal					
1	Pressing Hydraulic Foot Pedal will raise patient to the desired height. Call biomedical for assistance if unsure how to proceed.				
2	Pressing Down and Holding Hydraulic Foot Pedal will lower device.				
3	Tilt control handles are located at both ends, and control tilt position by use of Gas Springs.				
4	Engage Gas Springs to tilt device to a maximum of 4 degrees towards drain/foot end to drain.				
Cleaning the EZ2000 Shower Trolley					
1	Clean completely sealed cushions and frame using facility approved disinfectant.				
2	Rinse Trolley. Dry before next use.				
3	Wipe underside of Trolley to prevent water hazard.				
4	Do not autoclave cushions.				