



EZ Way, Inc.

# EZ Way Classic Lift Competency Quiz

## Questions

Question Group 1

**1. The battery has been discharged and needs to be charged or replaced when?**

*(1 point)*

- (A) The control box beeps when the up or down button is pressed
- (B) The LED on the control box turns red
- (C) The LED on the hand control turns yellow when the up or down button is pressed
- (D) A & C

**2. The hand control is the only method to raise/lower the resident/patient under normal operation?**

*(1 point)*

- True
- False

**3. Pressing the emergency stop button will immediately stop the machine from moving.**

*(1 point)*

- True
- False

**4. How do you manually lower a patient if the machine stops working?**

*(1 point)*

- Use scissors to cut the sling straps
- Turn the red collar on the shaft of the actuator clockwise
- Push the emergency stop button
- Use the hand control

**5. If the sling shows signs of tearing or excessive wear it is okay to use just once more.**

*(1 point)*

- True
- False



EZ Way, Inc.

# EZ Way Classic Lift Competency Quiz

**6. You should lock the wheels on the lift when raising a patient.**

*(1 point)*

- True
- False

**7. The top and the bottom of the sling should be positioned:**

*(1 point)*

- At the waist and the ankles
- Around the torso and outside of the arms
- At the patient's shoulders and tailbone
- Anywhere the patient wants it.

**8. You should cross the legs of the sling.**

*(1 point)*

- True
- False

**9. The sling cannot be removed from the patient when seated.**

*(1 point)*

- True
- False

**10. You should attach the sling to the lift with the boom above the ankles.**

*(1 point)*

- True
- False



EZ Way, Inc.

# EZ Way Classic Lift Competency Quiz

**11. Which of the following is a use for the straps on the back of the sling?**

*(Multiple Choice Question, 1 point)*

- Decoration
- To attach the sling to the lift
- To position the patient in a chair
- All of the above

**12. You should always use the longest loops of the sling.**

*(True/False Question, 1 point)*

- True
- False