



Reusable Glide™ Lateral Transfer System

Skills Checklist

Date: _____

Name: _____

Department: _____

Please mark each category with an "X" when completed

| Task/Action | Complete | Initials |
|---|----------|----------|
| Identify the safety icons | | |
| Describe the safety check associated with each icon | | |
| Grab an assistant to help with the transfer | | |
| Ensure both surfaces are flat and level | | |
| Lay mat on one surface | | |
| Make sure brakes are set on both platforms | | |
| Ensure the patient is centered on the mat | | |
| Buckle centering straps loosely | | |
| Attach the hose to the mat and secure with the Hook and Loop strap | | |
| Plug in blower | | |
| Turn on the blower | | |
| Grab the BackSmart® extended pull handles | | |
| One caregiver guide the patient, the other caregiver step back and transfer patient | | |
| Turn off and unplug the blower | | |
| Detach the hose from the mat | | |
| Re-pack the transport tote | | |

Comments:
