



EZ Way, Inc.

EZ Way Wipeable Fabric Cleaning Instructions

STAIN RESISTANCE, CLEANING & DISINFECTION

EZ Way's wipeable fabric made with Sure-Chek® healthcare fabrics were designed to be safe, comfortable, and long lasting in a healthcare setting. Sure-Chek healthcare fabrics resist staining from most medications, oils, grease and contaminants and the Sure-Chek antimicrobial fabric protection system performs for the life of the fabric. To maximize the life of your EZ Way accessories, EZ Way recommends the manufacturers regular cleaning guidelines:

GENERAL CLEANING:

- Soils or Stains: Use neutral soap suds and lukewarm water. Do not use harsh cleansers, solvents or detergents.
- Hard-to-Clean Spots: Use standard household/vinyl cleansers and a soft bristle brush and gentle agitation on troublesome spots or stains. Pre-soak heavy, dried-on soil.

DISINFECTING:

- Phenolic disinfectants are the best choice for vinyl products, though properly diluted quaternaries are also acceptable. Phenolic should be avoided on urethane.
- Quaternary disinfectants are recommended for urethane products. Quaternary/Isopropyl disinfectants are recommended for urethanes, but not recommended for vinyl.
- All EZ Way wipeable accessories made with Sure-Chek fabrics may be cleaned with a 1:10 dilution of household bleaches containing 5.25% sodium hypochlorite as recommended by the Center for Disease Control in Atlanta, Georgia. There is no harmful effect on the fabric when used in this dilution ratio.

SPECIFIC COMMERCIAL DISINFECTANTS:

- There are many popular disinfectants being used in care facilities. Cavi-Wipes and Virox5 are two of these that have been tested and found to be acceptable for use with Sure-Chek Healthcare fabrics when used according to manufacturer's recommendations.

LAUNDERING:

- Polyurethane-coated Sure-Chek Fabrics: Machine-wash with detergent up to 120°. (Some surface wrinkling may occur. This wrinkling has no adverse effect on the fabric properties). Hang or tumble dry thoroughly at 140° before storage.

PRECAUTIONS:

- Disinfectants applied at full concentration or in highly concentrated solutions may damage the fabric and will decrease the useful life of Sure-Chek fabrics.
- Iodophor type disinfectants used on EZ Way wipeable accessories made with Sure-Chek fabrics may result in staining. If stains from Iodophor (i.e. iodine bearing) type disinfectants are not treated with a diluted (10 to1) bleach solution within 20 minutes of application or spillage, staining will occur.
- Storage of fabrics that have not been dried thoroughly before folding and storing can promote the growth of mildew.

LIST OF CHEMICALS TESTED:

Believed to be safe to use on Sure-Chek brand healthcare fabrics when used according to manufacturer's directions

- **Virox 5 Cleaner** – Active Ingredient: 0.5% Hydrogen Peroxide
- **Oxivir® TB Disinfectant Cleaner** - Active Ingredient: 0.5% Hydrogen Peroxide
- **CaviWipes Disinfecting Towels** - Active Ingredient: 0.28% Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride; 17.2% Isopropanol
- **ProCure V** – Active Ingredient: Chlorine Dioxide
- **PDI Sani-Cloth HB Germicidal Disposable Wipe** - Active Ingredients: Alkyl dimethyl benzyl ammonium chloride and Alkyl dimethyl ethyl benzyl ammonium chloride
- **Clorox Healthcare Bleach Germicidal Wipes** - One-step, ready-to-use disinfectants - Active ingredient: 0.5% sodium hypochlorite

DISCLAIMER

This guide is designed to provide appropriate technical information and advice to fabricators and end-users. The information is given in good faith, but because of the many particular factors that are outside our knowledge and control, and affect the use of products, no warranty is expressed or is implied on its accuracy.